


Product Installation Guidelines & Scope of Use

Version 1, 8 August 2025, Page 1 of 3
Document No: 001.20.20.00

GalvinAssist® Folding Grab Rails with Fixed Toilet Roll Holder

PRODUCT CODES

- 105.62.08.31 (Left Hand)
- 105.62.08.41 (Right Hand)

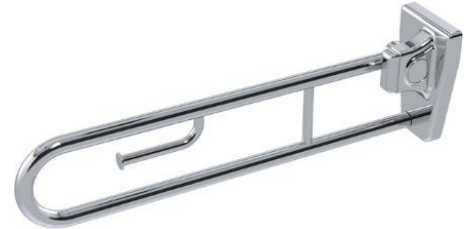

PRODUCT INFORMATION

- The GalvinAssist® range of Stainless Steel folding grab rails provides safe and reliable fixtures for WC's and accessible areas in health institutions, hospitals, aged care facilities and nursing homes.

WARNINGS: Special attentions to be paid on notes, photos, images, or drawings of assembly steps marked with the warning symbol.



TECHNICAL DATA

Features:	<ul style="list-style-type: none"> - Includes factory fitted stainless steel toilet paper holder. - Locks in horizontal or vertical positions. - Chrome plated brass push button release, base housing and concealed "T" mount bracket - Safe working load up to 200 kg (Tested by ANTL – test report 4197.12-22.07.13)
Fixings:	Rails are supplied with stainless steel fixing screws
Finish:	Satin Stainless Steel
NOTE: Galvin Engineering continually strive to improve their products. Specifications may change without notice.	


MOUNTING DETAILS

MUST BE FITTED TO A SOLID WALL OR OTHER SURFACE.


Grade 304 Stainless Steel M8 x 75mm hexagon coach screw and 10mm wall plugs supplied.

For fixing to timber, use only hexagon coach screws.

For fixing to brickwork, block work or concrete use the hexagon coach screw with the 10mm wall plugs or equivalent.


 **WARNING:** Install using stainless steel 304 or 316 screws. Other types of screws may rust and void warranty.

DIMENSIONS

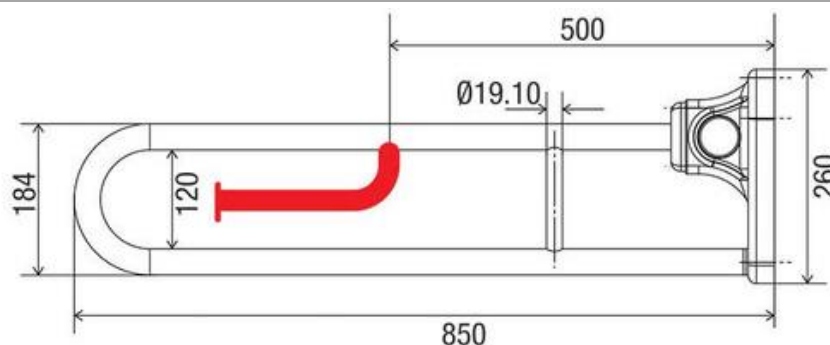


INSTALLATION

INSTALLATION COMPLIANCE: Galvin Engineering products must be installed in accordance with these installation instructions and in accordance with AS/NZS 3500, the PCA and your local regulatory requirements. Water and/or electrical supply conditions must also comply to the applicable national and/or state standards. Failing to comply with these provisions shall void the product warranty and may affect the performance of the product.

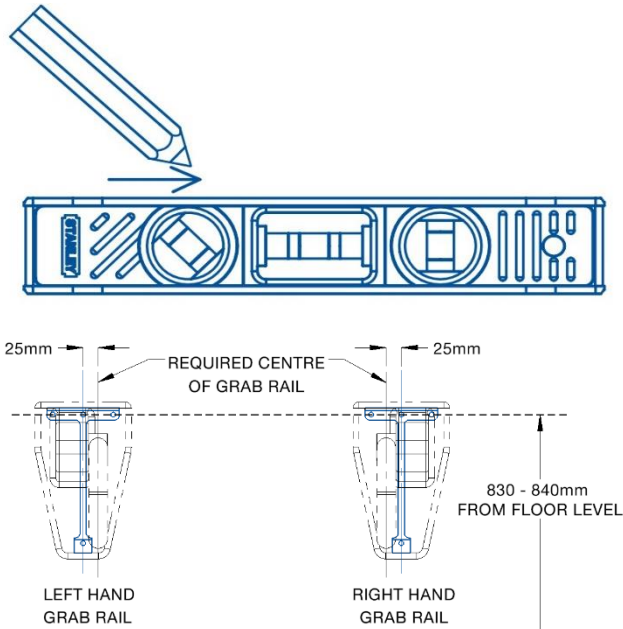

1. Position Wall Mount


- Mark horizontal line on wall 830-840mm from floor. Position wall mount with top fixing holes on horizontal line.
- Offset centre holes 25mm of required centre of grab rail (**For Left Hand grab rail, offset 25mm to the right. For Right Hand grab rail, offset 25mm to the left.** ⚠️)

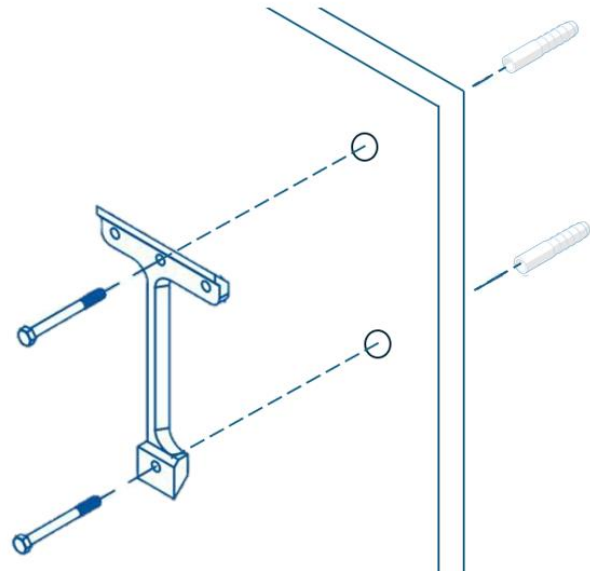

2. Fix Wall Mount


- Screw wall mount to wall with stainless steel coach screws supplied.

⚠️ Note: On stud wall, ensure wall mount is fixed into studwork and noggins. For masonry walls, use wall plugs supplied.

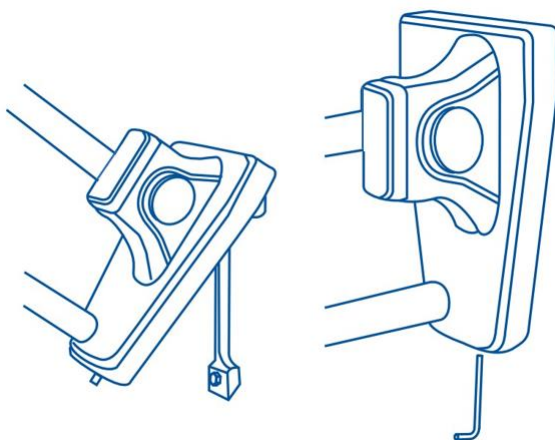

3. Fix Folding Grab Rail to Wall Mount

- Place folding grab rail over the wall mount and fasten with grub screw to secure.

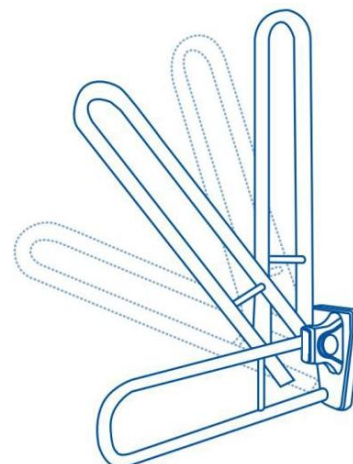

4. Test for correct operation

- Press release button to raise or lower grab rail.

WARRANTY

Galvin Engineering products are covered under our Manufacturer's Warranty. Galvin Engineering products must be installed in accordance with the installation instructions and in accordance with AS/NZS 3500 and NCC Volume Three, relevant Australian Standards and local authorities applicable to product being installed. Water and electrical supply conditions must also comply to the applicable national and/or state standards, failing to comply with these provisions may void the product warranty and affect performance of the product.

Please visit www.galvinengineering.com.au to view the full warranty, our Installation Compliance and Maintenance & Cleaning information as well as any other additional information.

Within Australia: **1300 514 074** Outside Australia: P: **+61 (0)8 9338 2344**

F: **+61 (0)8 9338 2340**

sales@galvinengineering.com.au

www.galvinengineering.com.au

ABN: 78 008 719 382

PERTH | SYDNEY | MELBOURNE | BRISBANE | ADELAIDE

